

**TOMTEC Imaging Systems GmbH**  
Freisinger Str. 9, 85716 Unterschleissheim, Germany

**UNTERSCHLEISSHEIM,**  
17.05.2021

## Vulnerability Disclosure Information 2021-0001

Dear Valued Customer,

Following a responsible disclosure process to continuously maintain the security of our products, we would like to inform you about some minor vulnerabilities within our software that we have become aware of.

### **CVE-2019-12741**

This vulnerability, reported in the "National Vulnerability Database" database (<https://nvd.nist.gov/>), concerns a third-party software which is integrated into **TOMTEC-ARENA TTA2.3x.xx**. You can find information about which version of TOMTEC-ARENA you are using in the About Box dialog.

#### **Who is affected?**

You could be affected, if you are using TOMTEC-ARENA within an unsecure environment (e.g. it is not protected from unauthorized access physically or remotely by using for example an up-to-date firewall or any other means).

#### **How can this vulnerability affect you?**

In case someone gets access (remote or physically) to your network and is able to directly inject malicious code into the TOMTEC-ARENA front end or uses social engineering techniques to mislead a user into opening a malicious link on the machine TOMTEC-ARENA is installed on, the attacker could gain unauthorized access to TOMTEC-ARENA.

#### **What can I do if I am not protected?**

The vulnerability has already been fixed with TOMTEC-ARENA TTA2.40.00. Thus, an update to **TOMTEC-ARENA TTA2.40.00 (or later)** will resolve this vulnerability.

In general you should always run TOMTEC-ARENA on a secure system with exclusive access rights for authorized staff. Furthermore your staff should be aware of the general security risk when opening links or attachments provided by uncertain sources.

## **CVE-2020-8112**

This vulnerability, reported in the “National Vulnerability Database” database (<https://nvd.nist.gov/>), concerns a third-party software which is integrated into **TOMTEC-ARENA TTA2.31.xx, TOMTEC-ARENA TTA2.40.xx and TOMTEC-ARENA TTA2.41.xx**. You can find information about which version of TOMTEC-ARENA you are using in the About Box dialog.

### **Who is affected?**

You could be affected, if you are using TOMTEC-ARENA within an unsecure environment (e.g. it is not protected from unauthorized access physically or remotely by using for example an up-to-date firewall or any other means).

### **How can this vulnerability affect you?**

In case someone gets access (remote or physically) to your network and is able to send a manipulated JPEG contained in a DICOM file to TOMTEC-ARENA, the attacker could cause a buffer overflow that may result in leakage of sensitive information.

### **What can I do if I am not protected?**

You should always run TOMTEC-ARENA on a secure system with exclusive access rights for authorized staff only to prevent anyone from being able to send manipulated DICOM files within your network.

The vulnerability will be fixed with **TOMTEC-ARENA TTA2.42.00 (or later)**, scheduled to be released end of July 2021.

## **CVE-2020-17510**

This vulnerability, reported in the “National Vulnerability Database” database (<https://nvd.nist.gov/>), concerns a third-party software which is integrated into **TOMTEC-ARENA TTA2.31.xx, TOMTEC-ARENA TTA2.40.xx and TOMTEC-ARENA TTA2.41.xx**. You can find information about which version of TOMTEC-ARENA you are using in the About Box dialog.

### **Who is affected?**

You could be affected, if your server running TOMTEC-ARENA is used within an unsecure environment (e.g. it is not protected from unauthorized access physically or remotely by using for example an up-to-date firewall or any other means).

### **How can this vulnerability affect you?**

In case someone gets access (remote or physically) to your network, the person could send a manipulated http request to the TOMTEC-ARENA server that may lead to the acquisition of unauthorized access to TOMTEC-ARENA.

### **What can I do if I am not protected?**

You should always run TOMTEC-ARENA on a secure system with exclusive access rights for authorized staff only.

The vulnerability will be fixed with **TOMTEC-ARENA TTA2.42.00 (or later)**, scheduled to be released end of July 2021.

Please contact TOMTEC Service & Support ([support@tomtec.de](mailto:support@tomtec.de)) in case you have questions or wish an update.

Kind regards

**GREGOR MALISCHNIG**

**CMO**